

ESI Adhesive Remover

Enzyme Solutions Adhesive Remover is a specially-formulated, ready-to-use product for the removal of ID tape and adhesive from medical equipment and apparatus. Safe for use on a variety of metals, glass, plastic and rubber.

For quick removal of stickers, labels, tape residue, adhesives, oil, grease, candle wax, chewing gum, ink/markers, glue, etc. Excellent for use on surgical instrument cases/trays, basins, case carts, autoclaves and surgical instruments. Neutral pH formula with surfactants and corrosion inhibitors.



DIRECTIONS FOR USE:

For label or tape removal:

1. Apply Adhesive Remover liberally on label or tape and around all edges
2. Allow to soak for 3-5 minutes.
3. Lift corner and peel back slowly

For tape residue:

1. Apply liberally
2. Allow to soak for 2 minutes
3. Wipe clean with dry cloth
4. Rinse item or put through normal cleaning process.

PRACTICAL TREATMENT FIRST AID EYES:

In case of contact, immediately flush eyes with plenty of water for at least 15 minutes. See a physician.

SKIN: Wash with soap and plenty of water for at least 15 minutes while removing contaminated clothing.

If irritation develops, seek medical aid. **INGESTION:** If swallowed, do not induce vomiting. Give water to dilute. Call a physician. Never give anything by mouth to an unconscious person.

PACKAGING:

- 12x8 Ounces

ENZYME SOLUTIONS®
INCORPORATED

2105 Forrest Park Drive
Garrett, IN 46738
800-523-1323 Phone • 260.497.0512 Fax
www.enzymesolutions.com